

QA4™

Surgical Versatility Guide



The complete surgical versatility guide for
your QA4™ Mobile Surgery System

AneticAid® 

SURGICAL VERSATILITY GUIDE

Contents

SURGICAL VERSATILITY GUIDE	
BREAST SURGERY	3-4
Supine / Fowler	
ENT (Ear, Nose and Throat)	5-7
Supine	
GENERAL SURGERY	8-11
Supine / Prone / Trendelenburg / Reverse Trendelenburg	
GYNAECOLOGY & OBSTETRICS	12-15
Supine / Trendelenburg	
HEAD AND NECK / OMFS	16-19
Supine	
OPHTHALMOLOGY	20-23
Supine	
ORTHOPAEDIC SURGERY	24-31
Supine / Fowler / Prone / Lateral	
UROLOGY	32-35
Supine / Trendelenburg	
VASCULAR SURGERY	36-39
Supine	
OPTIONAL EXTRAS	40-41
CLAMPS	42
QA4 ACCESSORY STAND	43

At Anetic Aid, our mission is to deliver the best outcomes for patients and best experiences for clinical teams. We listen carefully and understand needs.

This allows us to design, develop and manufacture, as well as establishing partnerships, that lead to the supply of innovative, intuitive, high-quality medical devices for patient transport, treatment and recovery.

The highly versatile QA4™ Mobile Surgery System evidences this approach, and the following pages detail the attachments and accessories which can be used to customise it for all main surgical specialisms.

They range from simple arm and hand tables to sophisticated dual-articulating headrests, and from lithotomy stirrups to orthopaedic fixation systems (for procedures such as total hip replacement).

This guide covers the options for ordering accessories as part of a QA4™ build specification and availability to purchase separately as stand-alone items.



BREAST SURGERY

- Positioning:
- Supine
- Fowler
- Prone
- Trendelenburg
- Reverse Trendelenburg
- Lateral



Breast surgery encompasses procedures performed on the breasts, ranging from cancer treatment like lumpectomy and mastectomy to cosmetic procedures like breast augmentation and reduction.

Many breast-conserving surgeries, sentinel lymph node biopsy, and some mastectomy procedures are now commonly performed as day cases.

Given the predicted increase in breast cancer incidence over the next thirty years, surgical management will place an unsustainable demand on in-patient services unless a significant shift to the day surgery setting is undertaken for appropriate procedures.

Some of the most performed breast procedures include:

- Simple mastectomy (without axillary surgery)
- Wide local excision of breast, including wire-guided
- Simple mastectomy with axillary surgery
- Excision of breast with sentinel lymph node biopsy, axillary sample or axillary clearance.



ACCESSORIES FOR BREAST SURGERY



Reison Flexible Positioning Armboard

The Flexible Positioning Armboard, by Reison Medical AB, is designed to provide stable support in a wide range of positions — including supine, prone, lateral, fowler and beach chair.

Description	Code
Flexible Rotating Armboard - with Adjustable Plate (20-300-10) - Integrated Clamp for UK EU & US Side Bar - includes Standard Pad	10400



Reison Rotating Armboard

Operated via a highly secure one-handed joint release mechanism, this armboard offers accurate positioning, patient comfort and excellent surgical access.

Description	Code
Armboard - Rotatable (10-387) - with Rotating Clamp - for EU Side Bar	10400-EU
Armboard - Rotatable (10-287) - with Rotating Clamp - for UK Side Bar	10400-UK
Armboard Cushion / Pad (10-382-H) - Tapered Bottom with Guttered Top Surface	10415-B1



Oasis Extremity Belt

Supplied in pairs, the Oasis Extremity Belt is used in many procedures for extra security in maintaining the position of extremities on the operating table.

Description	Code
Oasis Extremity Belt (OA113) - 315 x 45 x 10mm	10843

ENT (EAR, NOSE & THROAT)

Positioning: **Supine** Fowler Prone Trendelenburg Reverse Trendelenburg Lateral



ENT surgery deals with the surgical and medical management of conditions affecting the ears, nose, throat, and head and neck.

ENT surgeons diagnose, treat, and manage problems related to these areas, including hearing, balance, breathing, eating, and speech.

In general, ENT surgery is well suited to day surgery. Many of the procedures have a short operating time, with minimal post-operative morbidity and a low rate of complications. Most of the patients are of low anaesthetic risk with little co-morbidity.

Some of the most performed ENT procedures include:

- Tonsillectomy (paediatric)
- Tonsillectomy (adult)
- Diagnostic endoscopic examination of pharynx / larynx +/- biopsy
- Septoplasty of nose.



BUILD OPTIONS FOR ENT



NEW

ENT Headrest - Dual-Articulating with EU Side Bar

Our newly introduced dual-articulating head positioner - available as a build option and developed in conjunction with ENT surgeons - offers a wider operating surface and it's adaptable positioning helps to enable a wider range of surgery for ENT along with Oral & Maxillofacial.

Description	Code
ENT Headrest - Dual-Articulating with EU Side Bar	22332



Headrest - Full Width Articulating

Add even greater surgical versatility to your QA4™ Mobile Surgery System with this full width headrest, which is the full width of the stretcher - enabling a larger operating surface width overall.

Description	Code
Headrest - Full Width Articulating - with Pressure Care Mattress	22322



Compact Pushing Handles

Need even greater surgical access at the head-end on your QA4? Have your mobile surgery system supplied without the compact pushing handles fitted. Only available on code 21300 - QA4™ Mobile Surgery System - with Powered Functions.

Description	Code
Compact Pushing Handles	22345



NEW

ENT Headrest - Dual-Articulating with EU Side Bar

Our newly introduced dual-articulating head positioner - developed in conjunction with ENT surgeons - offers a wider operating surface and it's adaptable positioning helps to enable a wider range of surgery for ENT along with Oral & Maxillofacial.

Description	Code
ENT Headrest - Dual-Articulating with EU Side Bar	21323



Oasis+ Head & Neck Support

The Oasis+ Head & Neck Support incorporates silicone gel technology and is popular in ENT for septoplasty, some laryngoscopy cases and thyroidectomy. It also has anti-slip properties for additional safety.

Description	Code
Oasis+ Head & Neck Support (OP151) - 264 x 220 x 64mm	10900



Oasis Elite Open Headrings

Crafted from a unique gel and foam combination, the Oasis Elite Open Headrings gently conform to the contours of the patient's face, and the open-centre design allows for safe intubation.

Description	Code
Oasis Elite Open Headring (EL021) - Ø140 x 34mm	10863
Oasis Elite Open Headring (EL022) - Ø200 x 50mm	10864



Whitlock Horseshoe Headrest

Our popular, malleable open headrest, which includes a sculpted recess to alleviate pressure on the neck during head & neck procedures.

Description	Code
Whitlock Horseshoe Headrest	10185



GENERAL SURGERY

- Positioning:
- Supine
- Fowler
- Prone
- Trendelenburg
- Reverse Trendelenburg
- Lateral



What is general surgery?

General surgery is a broad surgical speciality focused on the alimentary canal and abdominal contents, including the oesophagus, stomach, intestines, liver, pancreas, gallbladder, and appendix, as well as the thyroid gland and other related areas.

The laparoscopic approach for cholecystectomy gained widespread acceptance in the early 1990s and was the major factor allowing the procedure to be performed as a day case. It is now considered the preferred technique by most surgeons for most indications.

Some of the most performed general surgery procedures include:

- Laparoscopic cholecystectomy
- Primary repair of inguinal hernia
- Incision and drainage of skin abscess
- Diagnostic laparoscopy




QA4™ Split Leg Positioner

Customise your QA4™ with this pair of individually removable, multi-position leg sections that include full length EU sized accessory side bar. The Split Leg Positioner enables the use of the QA4™ for a number of surgical procedures within such specialities such as gastro-oesophageal, bariatric and orthopaedics - without the need to pre-prepare the QA4™ with them attached, or the need to transfer the patient.

Description	Code
QA4™ Split Leg Positioner - with EU Side Bar*	21359

*Requires 2 x Clamps (Code: 10250)

ACCESSORIES FOR GENERAL SURGERY

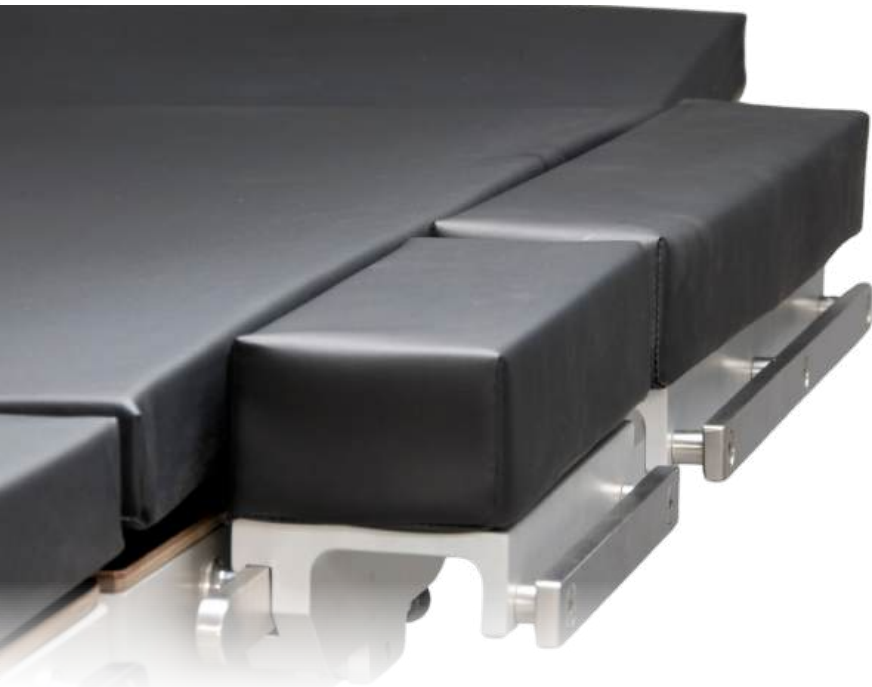

Operating Table Widening Sections

A product by Reison Medical AB, these sections widen the upper or lower torso of an operating table to accommodate larger patients.

Description	Code
Operation Table Widening Section - for EU Side Bar (10-421) - L240 x W125 x D85mm	10450-EU-A1
Operation Table Widening Section - for EU Side Bar (10-425) - L300 x W125 x D85mm	10450-EU-A2
Operation Table Widening Section - for EU Side Bar (10-422) - L445x W125 x D85mm	10450-EU-A3



ACCESSORIES FOR GENERAL SURGERY



Shoulder Supports

Specially designed to secure the patient in Trendelenburg position, these shoulder supports ensure patient protection without direct pressure on the brachial plexus, protecting against nerve damage


Description	Code
Shoulder Supports - with Fitted Pads	10787
Shoulder Supports - Replacement Pads (Left & Right)	10706



Security Strap

A product from Baxter®, these security straps loop around the side rail and adjust easily with buckles.

Description	Code
Security Strap - Medium - includes Padded Centre Section	10792
Security Strap - Large - includes Padded Centre Section	10793

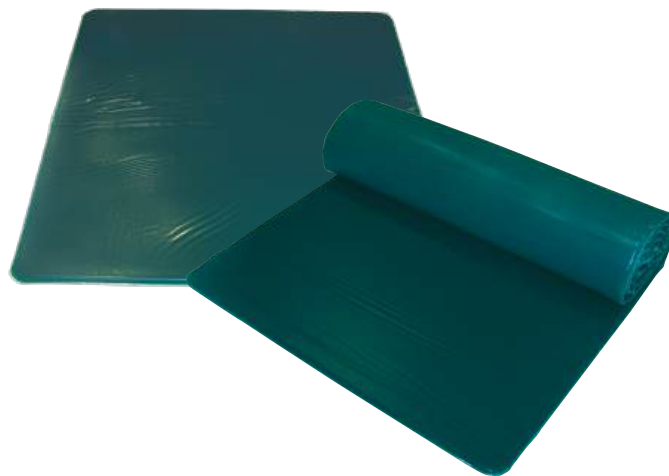

Oasis Standard Operating Table Pads

The Oasis Standard Operating Table Pads, measuring a range of sizes, incorporate silicone gel technology and provide protection over particular areas of the operating table, ranging from a universal size to the entire table. The pads are flexible, soft, comfortable and used with patients in all positions.

Description	Code
Oasis Standard Operating Table Pad (OA030) - Universal - 520 x 520 x 10mm	10807
Oasis Standard Operating Table Pad (OA031) - 3/4 Length - 1150 x 520 x 10mm	10808
Oasis Standard Operating Table Pad (OA032) - Full Length - 1800 x 520 x 10mm	10809

NEW



Vacuum Positioners

These vacuum filled beanbag positioners are regularly used to stabilise a patient's position on the operating table. We offer a range of shapes and sizes from full body to head stabilisers.

Description	Code
Vacuum Positioner - 4FORM - BODY - L1000 x W750mm	10275-A1
Vacuum Positioner - 4FORM - HEAD - L300 x W400mm	10278-A1
Vacuum Positioner - 4FORM - FULL LENGTH - L2000 x W750mm	10279-A1

ACCESSORIES FOR GENERAL SURGERY

Lift-Assist™ Stirrups - Standard

Our standard stirrup by Baxter® allow for easy adjustment of abduction and lithotomy while maintaining the sterile field.

Description	Code
Lift-Assist™ Lithotomy Stirrups - Standard - includes boot pads	*10755
*Requires 2 x Clamps (Code: 10761)	



Lift-Assist™ Stirrups - Finned Elite

The Finned Elite stirrups offer a 'fin'-shaped support at the knee and provide greater control when manoeuvring patient weights up to 227kg.

Description	Code
Lift-Assist™ Lithotomy Stirrups - Finned Elite - includes boot pads	*10757
*Requires 2 x Clamps (Code: 10761)	



Lift-Assist™ Stirrups - Heavy Duty

Easily accommodating patients with larger calves, the Heavy Duty stirrups are an upgrade to the standard - and are for patient weights up to 327kg.

Description	Code
Lift-Assist™ Lithotomy Stirrups - Heavy Duty - includes boot pads	*10758
*Requires 2 x Clamps (Code: 10761)	



GYNAECOLOGY & OBSTETRICS

Positioning: **Supine** | Fowler | Prone | **Trendelenburg** | Reverse Trendelenburg | Lateral



Gynaecological surgery involves procedures on the female reproductive system, addressing conditions like endometriosis, fibroids, cancer, and infertility, with options ranging from minimally invasive to open surgery.

Many common gynaecological procedures can be safely performed as day surgery, allowing patients to go home the same day, including laparoscopic hysterectomies, vaginal repairs, and various minor procedures.

Obstetric surgery encompasses surgical procedures performed on a pregnant woman for conditions related to pregnancy, labour, or the post-partum period; often involving procedures like caesarean sections or interventions for complications like haemorrhage or obstructed labour.

Some of the most performed gynaecology and obstetric procedures include:

- Termination of pregnancy
- Therapeutic endoscopic operations on uterus (including endometrial ablation)
- Therapeutic laparoscopic procedures including laser, diathermy and destruction e.g. endometriosis, adhesiolysis, tubal surgery
- Laparoscopy oophorectomy and salpingectomy (including bilateral)

ACCESSORIES FOR GYNAE & OBSTETRICS

Lift-Assist™ Stirrups - Standard

Our standard stirrup by Baxter® allow for easy adjustment of abduction and lithotomy while maintaining the sterile field.



Description	Code
Lift-Assist™ Lithotomy Stirrups - Standard - includes boot pads	*10755
*Requires 2 x Clamps (Code: 10761)	

Lift-Assist™ Stirrups - Heavy Duty

Easily accommodating patients with larger calves, the Heavy Duty stirrups are an upgrade to the standard - and are for patient weights up to 327kg.



Description	Code
Lift-Assist™ Lithotomy Stirrups - Heavy Duty - includes boot pads	*10758
*Requires 2 x Clamps (Code: 10761)	

Lift-Assist™ Stirrups - Finned Elite

The Finned Elite stirrups offer a 'fin'-shaped support at the knee and provide greater control when manoeuvring patient weights up to 227kg.



Description	Code
Lift-Assist™ Lithotomy Stirrups - Finned Elite - includes boot pads	*10757
*Requires 2 x Clamps (Code: 10761)	

Simplicity™ Knee Crutch

A product from Baxter, the Simplicity™ Knee Crutch provides excellent patient comfort and precise surgical positioning for short duration procedures.



Description	Code
Simplicity™ Knee Crutch - includes Boot Pads	10765
Boot Pads - for Simplicity™ Knee Crutch Stirrups	10766

Lithotomy stirrups with built-in patient protection



ACCESSORIES FOR GYNAE & OBSTETRICS



Shoulder Supports

Specially designed to secure the patient in Trendelenburg position, these shoulder supports ensure patient protection without direct pressure on the brachial plexus, protecting against nerve damage

Description	Code
Shoulder Supports - with Fitted Pads*	10787
Shoulder Supports - Replacement Pads (Left & Right)	10706

*Requires 2 x Clamps (Code: 10761)



Retrograde Extension

Need an even larger imaging window? This removable accessory adds an additional 240mm to create a 570mm imaging window with the QA4™ traversed fully to the foot end.

Description	Code
Urology / Retrograde Pyelography / Gynaecology Extension - with EU Side Bar	21397-EU

NOTE - ONLY FOR USE WITH CODE 21300



Perineal Instrument Tray

Our high quality, stainless steel, perineal instrument tray is ideal for gynaecology, urology and rectal procedures - fully compatible with our QA4™ Mobile Surgery System.

Description	Code
QA4 Perineal Instrument Tray - including locating pins	10235



Oasis & Oasis Elite Heel Pads

These 100% silicone Oasis Heel gel pads are supplied in pairs, and are suitable for use in any supine procedure where the heels are at risk of skin breakdown and damage. The Oasis Elite Heel pads combine foam and gel to provide maximum protection for the heels.

Description	Code
Oasis Heel Pads (OA130) - 180 x 100 x 70mm	10845
Oasis Elite Heel Pads (EL130) - 180 x 100 x 70mm	10875


ACCESSORIES FOR GYNAE & OBSTETRICS



Oasis Operating Table Pad - Torso Section with Cut Out

This Oasis Operating Table pad incorporates silicone gel technology and is specifically designed for use on the torso. Used on sectioned operating tables, this pad has a perineal cut-out, providing protection during surgeries such as orthopaedics and urology.

Description	Code
Oasis Operating Table Pad (OA037) - Torso Section with Cut-out - 900 x 500 x 10mm	10813

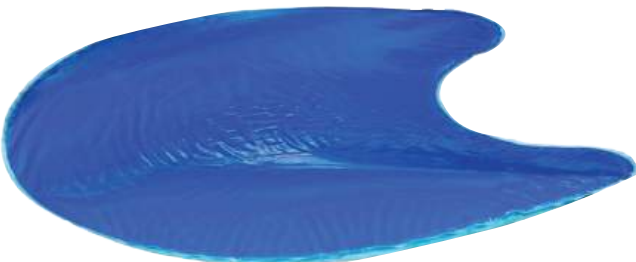

Azure Sacral Protector

Azure's revolutionary gel combination for the Azure Sacral Protector (AZ300) works together to replicate and act as an additional layer of protective tissue and skin, conforming to the individual's shape to protect against nerve damage as well as pressure sore prevention.

Description	Code
Azure Sacral Protector (AZ300) - 400 x 280 x 25mm	10888



HEAD AND NECK / OMFS

Positioning: **Supine** Fowler Prone Trendelenburg Reverse Trendelenburg Lateral



What encompasses “Head and Neck / OMFS” surgery?

Head and neck surgery, also known as otolaryngology-head and neck surgery, involves treating conditions affecting the ear, nose, throat, head, and neck, including both benign and malignant tumours, and reconstruction of the head and neck.

Oral and Maxillofacial Surgery (OMFS), often seen as the bridge between medicine and dentistry, is a surgical speciality that diagnoses and treats diseases, injuries, and irregularities of the mouth, jaws, face, and neck.

Some of the most performed head and neck / OMFS procedures include:

- Surgical removal of impacted / buried tooth/teeth (adult)
- Hemithyroidectomy, lobectomy, partial thyroidectomy
- Excision of lesion of parathyroids
- Excision of parotid gland

BUILD OPTIONS FOR HEAD AND NECK / OMFS



ENT Headrest - Dual-Articulating with EU Side Bar

Our newly introduced dual-articulating head positioner - available as a build option and developed in conjunction with ENT surgeons - offers a wider operating surface and it's adaptable positioning helps to enable a wider range of surgery for ENT along with Oral & Maxillofacial.

Description	Code
ENT Headrest - Dual-Articulating with EU Side Bar	22332



Headrest - Full Width Articulating

Add even greater surgical versatility to your QA4™ Mobile Surgery System with this full width headrest, which is the full width of the stretcher - enabling a larger operating surface width overall.

Description	Code
Headrest - Full Width Articulating - with Pressure Care Mattress	22322



Compact Pushing Handles

Need even greater surgical access at the head-end on your QA4? Have your mobile surgery system supplied without the compact pushing handles fitted. Only available on code 21300 - QA4™ Mobile Surgery System - with Powered Functions.

Description	Code
Compact Pushing Handles	22345



ACCESSORIES FOR HEAD & NECK / OMFS



NEW

ENT Headrest - Dual-Articulating with EU Side Bar

Our newly introduced dual-articulating head positioner - developed in conjunction with ENT surgeons - offers a wider operating surface and it's adaptable positioning helps to enable a wider range of surgery for ENT along with Oral & Maxillofacial.

Description	Code
ENT Headrest - Dual-Articulating with EU Side Bar	21323



Oasis+ Head & Neck Support

The Oasis+ Head & Neck Support incorporates silicone gel technology and is popular in ENT for septoplasty, some laryngoscopy cases and thyroidectomy. It also has anti-slip properties for additional safety.

Description	Code
Oasis+ Head & Neck Support (OP151) - 264 x 220 x 64mm	10900



Oasis Elite Open Headrings

Crafted from a unique gel and foam combination, the Oasis Elite Open Headrings gently conform to the contours of the patient's face, and the open-centre design allows for safe intubation.

Description	Code
Oasis Elite Open Headring (EL021) - Ø140 x 34mm	10863
Oasis Elite Open Headring (EL022) - Ø200 x 50mm	10864



Oasis Elite Closed Headrings

Containing a unique gel and foam combination, and 50% lighter than comparable gel-only products, the Oasis Elite Closed Headrings are suitable for use in many surgical procedures.

Description	Code
Oasis Elite Closed Headring (EL011) - Ø140 x 34mm	10861
Oasis Elite Closed Headring (EL012) - Ø200 x 50mm	10862



Rubens Pillow

This integral skin, polyurethane, foam moulded Rubens Pillow enables superb head stability whilst also providing a surface for surgical instruments during ENT and Ophthalmic procedures. It also allows the surgeon to rest their elbows on the pillows 'wings' - providing comfort for the patient and practitioner alike.

Description	Code
Rubens Pillow Adult	10180

ACCESSORIES FOR HEAD AND NECK / OMFS



Oasis Elite Supine Head Rests

The Oasis Elite Supine Head Rests incorporate a combination of silicone gel and foam technology, and are ideal for positioning the patient during eye surgery and other supine procedures.

Description	Code
Oasis Elite Supine Head Rest (EL151) - 275 x 237 x 90mm	10873
Oasis Elite Supine Head Rest (EL152) - 275 x 230 x 65mm	10874



Theatre Pillow

Tired of the number of pillows heading to the laundry, reducing their availability? This water-impermeable material-covered, superbly comfortable pillow reduces the turnaround time for pillows between uses.

Description	Code
Theatre Pillow	10545



OPHTHALMOLOGY

Positioning: **Supine** Fowler Prone Trendelenburg Reverse Trendelenburg Lateral



What is ophthalmic surgery?
Ophthalmic surgery, also known as ocular surgery, involves procedures on the eye or its surrounding tissues to treat various conditions, including cataracts, glaucoma, and refractive errors.
Many common ophthalmic surgeries, including cataract removal, can be performed as day surgeries, meaning patients are admitted and discharged on the same day, often under local anaesthesia.
Some of most performed ophthalmic surgeries include:

- Extraction of cataract +/- implant
- Vitrectomy using pars plana approach
- Dacryocystorhinostomy including insertion of tube
- Excision lesion of eyelid (adult)



Headrest - Dual Articulating

Replace the standard width removable articulating headrest with one that articulates not only on the head section, but also the neck as well. This innovative, intuitive and easy to use head positioner offers superb surgical positioning, surgical access and patient comfort: perfectly suited to ophthalmology procedures.

Description	Code
Headrest - Dual-articulating Head Positioner - with Shaped Cushion and Neck Plate Pad	22325

BUILD OPTIONS FOR OPHTHALMOLOGY



Leg Section - Articulating

Add even greater surgical versatility to your QA4™ Mobile Surgery System with this removable leg section that articulates to -45 degrees, enabling even greater patient comfort in recovery.

Description	Code
Leg Section - Articulating	22355



Compact Pushing Handles

Need even greater surgical access at the head-end on your QA4? Have your mobile surgery system supplied without the compact pushing handles fitted. Only available on code 21300 - QA4™ Mobile Surgery System - with Powered Functions.

Description	Code
Compact Pushing Handles	10807



ACCESSORIES FOR OPHTHALMOLOGY



ENT Headrest - Dual-Articulating with EU Side Bar

Our newly introduced dual-articulating head positioner - developed in conjunction with ENT surgeons - offers a wider operating surface and it's adaptable positioning helps to enable a wider range of surgery for ENT along with Oral & Maxillofacial.

Description	Code
ENT Headrest - Dual-Articulating with EU Side Bar	21323



Rubens Pillow

Ideal for ophthalmics, the Rubens Pillow provides secure positioning of the head while allowing the surgeon to rest their elbows on the pillows 'wings'.

Description	Code
Rubens Pillow - Adult	10180



Drape Screen / Oxygen Delivery Bar

Two products in one. An over patient drape / anaesthetic screen and oxygen delivery system. Mounted via a clamp on the side rail and flexible to be bent over the patient.

Description	Code
Oxygen Delivery Bar & Drape Screen	21365
Clamp - Universal (Socket & Rail) - Handwheel*	10250

*NOTE - CLAMP SOLD SEPARATELY



Headrest- Dual-Articulating

This innovative, intuitive and easy to use head positioner, that articulates on the head and neck sections, offers superb surgical positioning, surgical access and patient comfort: perfectly suited to ophthalmology procedures.

Description	Code
Headrest - Dual-Articulating Head Positioner - with Shaped Cushion and Neck Plat Pad	21325



Theatre Pillow

Tired of the number of pillows heading to the laundry, reducing their availability? This water-impermeable material-covered, superbly comfortable pillow reduces the turnaround time for pillows between uses.

Description	Code
Theatre Pillow	10545

ACCESSORIES FOR OPHTHALMOLOGY



Oasis+ Head & Neck Support

The Oasis+ Head & Neck Support incorporates silicone gel technology and is popular in ENT & Ophthalmics, and surgeries such as septoplasty, some laryngoscopy cases and cataract surgery. It also has anti-slip properties for additional safety.

Description	Code
Oasis+ Head & Neck Support (OP151) - 264 x 220 x 64mm	10900



Oasis Elite Supine Head Rests

The Oasis Elite Supine Head Rests incorporate a combination of silicone gel and foam technology, and are ideal for positioning the patient during eye surgery and other supine procedures.

Description	Code
Oasis Elite Supine Head Rest (EL151) - 275 x 237 x 90mm	10873
Oasis Elite Supine Head Rest (EL152) - 275 x 230 x 65mm	10874



ORTHOPAEDIC SURGERY

Positioning: **Supine** | Fowler | Prone | Trendelenburg | Reverse Trendelenburg | Lateral



What does orthopaedic surgery consist of?

Orthopaedic surgery is a medical speciality focused on the diagnosis, treatment, and prevention of injuries and diseases of the musculoskeletal system, including bones, joints, muscles, and associated soft tissues.

These procedures have traditionally been associated with in-patient admission, often for a duration of many days. However, several UK centres are now successfully performing both total hip and total knee replacement surgery as day case procedures.

Some of the most performed orthopaedic surgery procedures include:

- Examination / manipulation of joint under anaesthetic +/- injection
- Autograft anterior cruciate ligament reconstruction
- Removal of internal fixation from bone / joint, excluding K-wires
- Arthroscopy of knee including meniscectomy, meniscal or other repair

Leg Section with Side Bar

This removable leg section, complete with full length EU sized (25mm x 10mm) accessory side bar for the QA4™ Mobile Surgery System can replace the standard light weight leg section in this build option.

This option allows even greater surgical versatility for when an accessory needs to be attached in the vicinity of a lower limb.

Dimensions & Benefits

- **Weight:** 8.5kg
- **Dimensions-** L731 x W600mm
- **Detachable and lightweight** - the whole leg section is build to be light weight and can be detached from the rest of the QA4 with ease.
- **Durability & Reliability** - built for long-term and repeated use
- **Pressure care built in** - our leg sections, much like our full stretchers, come with a built in pressure care mattress - providing enhanced patient comfort.

Description	Code
Leg Section - Standard Light Weight - with Full Length EU Side Bar and Pressure Care Mattress	22358



Leg section build option to maximise surgical access



ACCESSORIES FOR ORTHOPAEDIC SURGERY



QA4™ Split Leg Positioner

Add even greater surgical versatility to your QA4™ with this pair of individually removable, multi-position leg sections that include full length EU sized accessory side bar. Useful for a number of surgical procedures including gastro-oesophageal, bariatric and orthopaedics.

Description	Code
QA4™ Split Leg Positioner - with EU Side Bar	21359



Retrograde Extension

Need an even larger imaging window? This removable accessory adds an additional 240mm to create a 570mm imaging window with the QA4™ traversed fully to the foot end.

Description	Code
Urology / Retrograde Pyelography / Gynaecology Extension - with EU Side Bar	21397-EU

NOTE - ONLY FOR USE WITH CODE 21300



Leg Section with Side Bar

This removable, standard light weight leg section includes full length EU sized (25mm x 10mm) accessory side bar, enabling greater surgical versatility for when an accessory needs to be attached in the vicinity of a lower limb.

Description	Code
Leg Section - Standard Light Weight - with Full Length EU Side Bar	21357E



The Woodlands Arm & Hand Surgery Table

Our specially designed surgery table offering more rigidity in the supine position

Developed in conjunction with a consultant hand surgeon, the Woodlands Arm & Hand Surgery Table is perfect when the patient is in a supine position and the surgery requires true rigidity.

Description	Code
The Woodlands Arm & Hand Surgery Table	21385

ACCESSORIES FOR ORTHOPAEDIC SURGERY



The Magna Hand Surgery Table

Supine, seated or partially elevated? The Magna Hand Surgery Table covers it all.

Developed in conjunction with a consultant hand surgeon, this hand table is perfect for all types of hand surgery when the patient in supine, seated or partially elevated positions.

Description	Code
The Magna Hand Surgery Table*	21384

*Requires 1 x Clamp (Code: 10248)



The Gardiner Hand Surgery System™

The perfect solution for high volume, low complexity hand surgery.

Developed in conjunction with consultant plastic & hand surgeon Matthew Gardiner and Frimley Health NHS Foundation Trust, this hand table is perfect for supporting hand surgery using the WALANT approach - with the patient in a comfortable seated position.

Description	Code
The Gardiner Hand Surgery System™*	21380

*Requires 2 x Clamps (Code: 10248-B1 / 10248-B2)



ACCESSORIES FOR ORTHOPAEDIC SURGERY



Lateral Brace 3-Piece

A product by Baxter®, this Lateral Brace comes with extended posts that adjust for use on a variety of patients. It stabilised patients on operating tables for total hip and other procedures requiring optimum lateral stability.

Description	Code
Lateral Brace - 3-piece Set - includes Two Rectangular Supports & One Circular Support - Fitted Pads*	*10770

*Requires 3 x Clamps (Code: 10761)



Reison Lateral Supports

This type of support includes two pads, one flat and one curved, to accommodate different patient physiques. It can also be fixed in a number of positions.

Description	Code
Lateral Support - Narrow (10-560) - Rotatable - includes Pads	*10430-L1
Lateral Support - Wide (10-561) - Rotatable - includes Pads	*10430-L2
Circular Pubic Support (10-567)	*10430-P1

*Requires 1 x Clamp (Code: 10250-EU / 10250-UK)



Montreal Laminectomy Support

The Montreal Laminectomy Support is a positioning device used in spinal procedures - and any other surgeries requiring the prone position. It's ergonomically designed to ensure maximum patient comfort.

Description	Code
Montreal Laminectomy Prone Positioner - LARGE	10160
Montreal Laminectomy Prone Positioner - MEDIUM	10161
Montreal Laminectomy Prone Positioner - EXTRA LARGE	10162



Lateral / Pelvic Supports

Our popular, quick, simple and easy to use lightweight lateral and pevic supports. Both supports are adjustable in height to allow for optimal and comfortable positioning.

Description	Code
Lateral Support - with Pad	*10200
Lateral Support - Pad ONLY	10201
Pelvic Support - with Pad	*10205
Pelvic Support - Pad ONLY	10206

*Requires 1 x Clamp (Code: 10250)

ACCESSORIES FOR ORTHOPAEDIC SURGERY



Total Knee Stabiliser

A covenient, stable device to support the flexed leg during total knee procedures. The Total Knee Stabiliser eliminates awkward sandbags and simply mounts to the table rail.

Description	Code
Total Knee Stabiliser - with Pad*	10785

*Requires 1 x Clamp (Code: 10250)




Positioner - D-Shaped

This ergonomically shaped domed positioner is made from a polyurethane coated material, polyurethane foam and is perfectly suited to positioning legs with the knee rasied.

Description	Code
Positioner - D-Shaped (Knee) - W550 x D300 x H175mm	10295



ACCESSORIES FOR ORTHOPAEDIC SURGERY

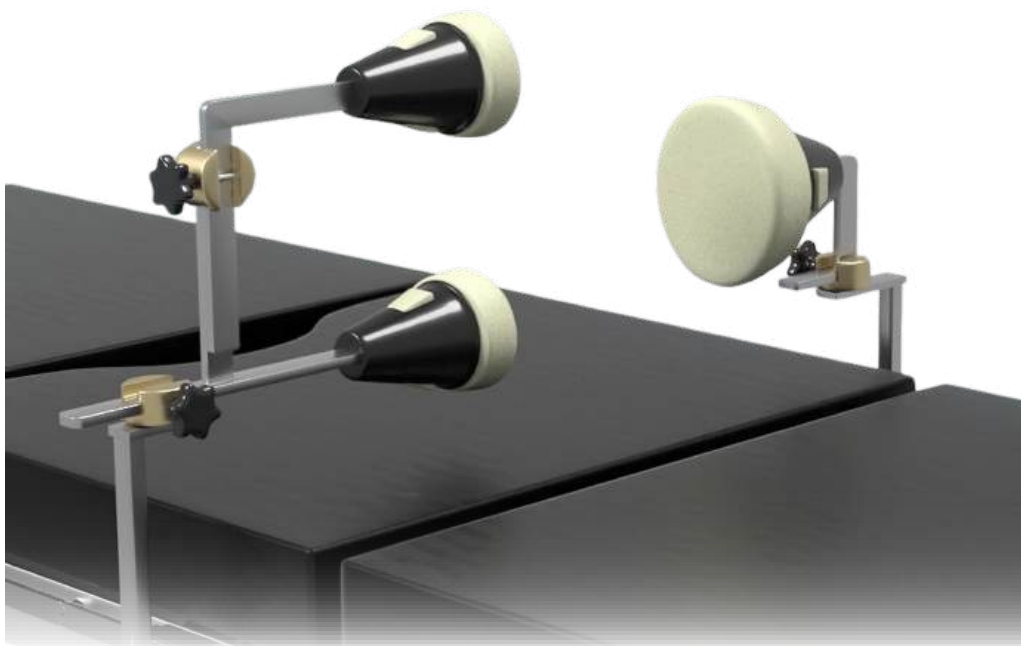

Hip Fix™ System

The Hip-Fix™, a product by Opitek, has a three-point fixation for the lateral positioning, securing both sides of the iliac crest plus the sacrum. This gives the surgeon maximum access while maintaining total stability.

Description	Code
Hip Fix™ (2015-200) 3-point Fixation System - uses disposable pads	10470
Hip Fix™ (2000-182) 3-point Fixation System - with fixed pads	10470-A1
Hip Fix™ (2000-182) 3-point Fixation System - individual support adjustment - disposable pads	10470-A2
Hip Fix™ (1900-01) 3-point Fixation System - individual support adjustment - fixed pads	10470-A3
Hip Fix™ Single-use Pad Set (2015-50) - includes one sacrum & two iliac crest positioner pads	10470-P1
Hip Fix™ Rotary Clamp (10-305-P) - Hook-on for UK Side Bar with Flat Pushing Plunge	10470-C1
Hip Fix™ Rotary Clamp (10-305-P) - Hook-on for EU Side Bar with Flat Pushing Plunge	10470-C2




Beach Chair

A product by Baxter®, the Lift-Assist™ Beach Chair is designed for use with orthopaedic procedures using the Fowler's or beach chair position. The chair offers unobstructed access and the Lift-Assist mechanism makes repositioning the patient quick and easy.

Description	Code
Lift-Assist™ Beach Chair - includes reusable pad set and intubation plate and pad	10786
Lift-Assist™ Beach Chair - Standard Head Positioner - includes pad set	10786-H0
Counter Traction Strap	10786-R1
Storage Cart - for Lift-Assist™ Beach Chair	10786-S1

ACCESSORIES FOR ORTHOPAEDIC SURGERY

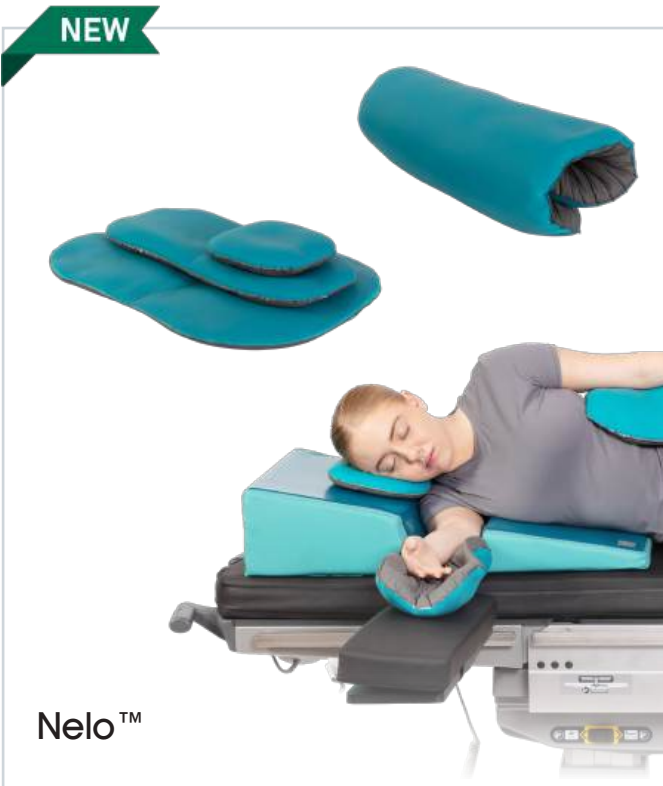

Nelo™

A brand-new product by Trulife, the Nelo™ is a versatile, dual-layer universal positioner. It can be manipulated to support any part of the body whilst still giving the stability, comfort, relief and neutral body positioning required.

Description	Code
Nelo - Universal Positioner - Small - (NE100) - 395 x 115 x 10mm	10275-A1
Nelo - Universal Positioner - Medium - (NE200) - 240 x 490 x 40mm	10278-A1
Nelo - Universal Positioner - Large - (NE300) - 440 x 590 x 40mm	10279-A1



Oasis Elite Prone Chest Positioner

The Oasis Elite Prone Chest Positioner combines gel and foam technology and has been designed to provide effective positioning to the patient during surgical procedures in the prone position.

Description	Code
Oasis Elite Prone Chest Positioner (EL903) - 610 x 530 x 130mm	10883



Elite Foam Prone Positioner - Montreal Laminectomy

This Elite Foam Prone Positioner is suited to relieving pressure on the abdomen during extended procedures in the prone position.

Description	Code
Elite Foam Prone Positioner (ELP914) - Montreal Laminectomy - 1014 x 480 x 140mm	10876



Oasis Elite Large Prone Positioner - Montreal Laminectomy

The Oasis Elite Large Prone Positioner eliminates pressure on the abdomen and facilitates easy ventilation and circulation during long cases in the prone position e.g. laminectomy procedures.

Description	Code
Elite Foam Large Prone Positioner (EL904) - Montreal Laminectomy - 1115 x 520 x 140mm	10990



- Positioning:
- Supine
- Fowler
- Prone
- Trendelenburg
- Reverse Trendelenburg
- Lateral



Urology surgery...Here’s what you need to know.

Urology surgery encompasses surgical procedures performed to treat conditions affecting the urinary system, male and female reproductive organs, and adrenal glands.

Many common urology surgeries, including procedures like ureteroscopy and cystoscopy, are now commonly performed as day surgeries, meaning patients can go home the same day.

Some of the most performed urology procedures include:

- Endoscopic resection / destruction of lesion of bladder
- Diagnostic endoscopic examination of bladder (including any biopsy)
- Endoscopic insertion of prosthesis into ureter
- Ureteroscopic extraction of calculus of ureter



Lift-Assist™ Stirrups - Standard

Our standard stirrup by Baxter® allows for easy adjustment of abduction and lithotomy while maintaining the sterile field. These stirrups are suitable for patient weights up to 159kg.

Description	Code
Lift-Assist™ Lithotomy Stirrups - Standard - includes boot pads	*10755

*REQUIRES 2 x CLAMPS (CODE: 10761)



Lift-Assist™ Stirrups - Finned Elite

The Finned Elite Stirrups offer a ‘fin’-shaped support at the knee and provide greater control when manoeuvring patient weights up to 227kg.

Description	Code
Lift-Assist™ Lithotomy Stirrups - Finned Elite - includes boot pads	*10757

*REQUIRES 2 x CLAMPS (CODE: 10761)



Lift-Assist™ Stirrups - Heavy Duty

Easily accommodating patients with larger calves, the Lift-Assist™ Heavy Duty Stirrups are an upgrade to the standard - and are for patient weights up to 327kg.

Description	Code
Lift-Assist™ Lithotomy Stirrups - Heavy Duty - includes boot pads	*10758

*REQUIRES 2 x CLAMPS (CODE: 10761)



ACCESSORIES FOR UROLOGY



Retrograde Extension

Need an even larger imaging window? This removable accessory adds an additional 240mm to create a 570mm imaging window with the QA4™ traversed fully to the foot end.

Description	Code
Urology / Retrograde Pyelography / Gynaecology Extension - with EU Side Bar	21397-EU

NOTE - ONLY FOR USE WITH CODE 21300



Perineal Instrument Tray

Our high quality, stainless steel, perineal instrument tray is ideal for gynaecology, urology and rectal procedures - fully compatible with our QA4™ Mobile Surgery System.

Description	Code
QA4 Perineal Instrument Tray - including locating pins	10235



Shoulder Supports

Specially designed to secure the patient in Trendelenburg position, these shoulder supports ensure patient protection without direct pressure on the brachial plexus, protecting against nerve damage

Description	Code
Shoulder Supports - with Fitted Pads*	10787
Shoulder Supports - Replacement Pads (Left & Right)	10706

*Requires 2 x Clamps (Code: 10761)



Simplicity™ Knee Crutch

A product from Baxter, the Simplicity™ Knee Crutch provides excellent patient comfort and precise surgical positioning for short duration procedures.

Description	Code
Simplicity™ Knee Crutch - includes Boot Pads	10765
Boot Pads - for Simplicity™ Knee Crutch Stirrups	10766



Oasis Operating Table Pad - Torso Section with Cut-out

This Oasis gel torso pad, incorporating silicone gel technology, is designed for us on sectioned tables. The torso section has a perineal cut-out - useful for orthopaedics and urology.

Description	Code
Oasis Operating Table Pad (OA037) - Torso Section with Cut-out 900 x 500 x 10mm	10813


ACCESSORIES FOR UROLOGY



Table Width Extenders

This table width extender increases the support surface for larger patients and can be used individually or in multiples to create a custom platform.

Description	Code
Table Width Extender - for UK Side Bar - includes Pad - 75mm	10788
Table Width Extender - Replacement Pad - 75mm	10788-P1

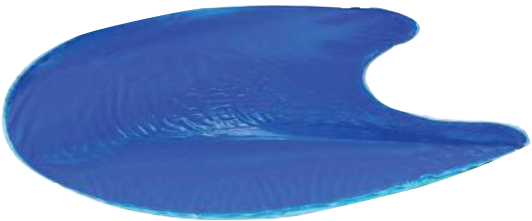

Azure Sacral Protector

The sacral area is an area of serious risk if left unprotected. The Azure Sacral Protector (AZ300) incorporates silicone gel technology and has been designed to provide support in the supine and lithotomy positions, relieving pressure and focusing on peripheral support.

Description	Code
Azure Sacral Protector (AZ300) - 400 x 280 x 25mm	10888



VASCULAR SURGERY

Positioning: **Supine** | Fowler | Prone | Trendelenburg | Reverse Trendelenburg | Lateral



What is vascular surgery?
Vascular surgery is a surgical speciality that deals with diseases affecting the arteries, veins, and lymphatic vessels. It addresses conditions such as aneurysms, peripheral artery disease, and varicose veins, often using minimally invasive techniques.
Depending on the procedure, some vascular surgeries can be performed as a day case procedure, without the need for in-patient admission.
Some of the most performed vascular surgeries include:

- Transluminal operations on iliac and femoral artery
- Creation of arteriovenous fistula for dialysis
- Varicose vein surgery
- Radiofrequency ablation of varicose veins (VNUS)

ACCESSORIES FOR VASCULAR SURGERY



Lift-Assist™ Stirrups - Standard

Our standard stirrup by Baxter® allows for easy adjustment of abduction and lithotomy while maintaining the sterile field. These stirrups are suitable for patient weights up to 159kg.

Description	Code
Lift-Assist™ Lithotomy Stirrups - Standard - includes boot pads	*10755



Lift-Assist™ Stirrups - Finned Elite

The Finned Elite Stirrups offer a 'fin'-shaped support at the knee and provide greater control when manoeuvring patient weights up to 227kg.

Description	Code
Lift-Assist™ Lithotomy Stirrups - Finned Elite - includes boot pads	*10757



Lift-Assist™ Stirrups - Heavy Duty

Easily accommodating patients with larger calves, the Lift-Assist™ Heavy Duty Stirrups are an upgrade to the standard - and are for patient weights up to 327kg.

Description	Code
Lift-Assist™ Lithotomy Stirrups - Heavy Duty - includes boot pads	*10758



ACCESSORIES FOR VASCULAR SURGERY

Split Leg Positioner

Add even greater surgical versatility to your QA4™ with this pair of individually removable, multi-position leg sections that include full length EU sized accessory side bar.

The Split Leg Positioner further enables the use of the QA4™ for a number of surgical procedures within such specialities as gastro-oesophageal, bariatric and orthopaedics - without the need to pre-prepare the QA4™ with them attached, or the need to transfer the patient.

Features:

- **Twist-grip handles**
- **Greater manoeuvrability and surgical access**
- **No pre-preparation needed**
- **Multi-purpose specialities**
- **Maximum patient weight: 160kg**



Description	Code
QA4™ Split Leg Positioner - with EU Side Bar	21359

*REQUIRES 2 x CLAMPS (CODE: 10250)



Azure Leg Protector

The Azure Leg Protector provides overall protection to the heel and leg area. Incorporating gel technology and slightly raised behind the knee, the leg protector offers protection to the popliteal nerve and conforms superbly around the leg.

Description	Code
Azure Leg Protector (AZ400) - 515 x 180 x 40mm	10892

ACCESSORIES FOR VASCULAR SURGERY



Azure Heel Protector

The heel area is one of extreme risk during any surgical procedure and special attention should be given to avoid any unnecessary damage. These lightweight, state-of-the-art gel protectors are supplied in pairs and provide heel protection, stability and support.

Description	Code
Azure Heel Protector (AZ401) - 515 x 180 x 40mm	10891



Azure Profile Pads

These Azure Profile Pads combine the ultra-soft and lightweight silicone material on the base and the cured silicone gel on top, with a unique squared profile that cushions the body during surgery.

Description	Code
Azure Profile Pad (AZ610) - Universal - 520 x 520 x 10mm	10886
Azure Profile Pad (AZ610) - 3/4 Length - 1150 x 520 x 10mm	10887



OPTIONAL EXTRAS - BUILD OPTIONS



Department ID Labelling

Struggling to identify a QA4™ as belonging to your department? Add your department name to your QA4 to aid equipment identification.

PLEASE NOTE: A MAXIMUM OF 22 CHARACTERS INCLUDING SPACES ARE ALLOWED.

Description	Code
Department ID Labelling - Vinyl Transfer Lettering	21175



Oxygen Cylinder Bracket

Add a compact and conveniently positioned bracket, retaining the position of CD sized oxygen cylinder brackets within the base mounted cylinder trough.

Description	Code
Oxygen Cylinder Bracket - for CD Sized Cylinder	21393

OPTIONAL EXTRAS - ACCESSORIES



Push / Pull Bar

This foot-end mounted removable accessory helps to cater for those scenarios where additional manoeuvring requirements arise.

Description	Code
Push / Pull Bar	21390



Side Rail Cover - GREY

Foam padded covers, with a grey print, are made with a hard wearing wipeable material & designed to completely encapsulate the QA4™ side rail when additional padding is felt to be needed.

Description	Code
Side Rail Cover - PADDED - for Single-piece Side Rail - GREY MATERIAL	21347



Foot-end Extension

Need extra leg room? Specifically designed for use with the QA4™, this foot-end mounted accessory adds an extra 300mm in length to the stretcher for taller patients.

Description	Code
Foot-end Extension - with 30mm Deep Mattress	21395

Side Rail Cover - CHILD PRINT

Foam padded covers, with a child-friendly print, are made with a hard wearing wipeable material & designed to completely encapsulate the QA4™ side rail when additional padding is felt to be needed.

Description	Code
Side Rail Cover - PADDED - for Single-piece Side Rail - CHILD PRINT MATERIAL	21348



Optional extras: from side rail covers to foot-end extensions



CLAMPS

Discover our huge range of clamps

We offer a full range of clamps, both direct-on and fixed, for UK and EU Side Bar.

Compatible with our QA4 Mobile Surgery System, these clamps provide security and stability for many different procedures - including gynaecology, urology, ophthalmology and many more!



Product-Specific Clamps

Our product-specific clamps go hand-in-hand with a range of products we supply, which you can find listed below:

- Gardiner Hand Surgery System (10248-B1 / 10248-B2)
- PediStirrup (10480-C1)
- Lift-Assist™ Beach Chair (10786-K2)
- Hip Fix™ System (10470-C1 / 10470-C2)

Description	Code
Clamp - Socket - Stainless Steel (10-308-UK) - Hook on with Lever Lock - UK Side Bar - V STUD	10248-B1
Clamp - Socket - Stainless Steel (10-307) - Hook on with Lever Lock - EU Side Bar - V STUD	10248-B2
PediStirrup Clamp - for EU & UK Side Bar	10480-C1
Clamp - for Lift-Assist™ Beach Chair - for EU Side Bar	10786-K2
Hip Fix™ Rotary Clamp (10-305-P) - Hook-on for UK Side Bar with Flat Pushing Plunge	10470-C1
Hip Fix™ Rotary Clamp (10-305-P) - Hook-on for EU Side Bar with Flat Pushing Plunge	10470-C2

QA4 ACCESSORY STAND

- Benefits:
- Adaptable
- Robust
- Multi-functional
- EU Side Bar

QA4™ Mobile Accessory Stand

QA4™ accessories need storage when not in use - and we have the perfect solution.

Our QA4 Mobile Accessory Stand, complete with EU side bar, is a compact, mobile storage system which helps to keep a busy theatre environment tidy and uncluttered.

Features & Benefits:

- Dimensions: W730 x D720 x H1375mm
- Weight - 75kg
- Compatible with a huge range of QA4 accessories
- Complete with EU Side Bar
- Robust, adaptable and multi-functional

Description	Code
QA4™ Mobile Accessory Stand - with EU Side Bar	21370



SEE
ALSO



QA4™
Mobile Surgery System

The complete multi-purpose solution for transport, treatment and recovery.

AneticAid

QA4 Mobile Surgery System

The complete multi-purpose solution for transport, treatment and recovery.

PLUS
our other product areas



Scan this code to see our **QA4's** journey through day surgery!



QA3™
Ophthalmic Stretcher



Outstanding surgical access – exceptional patient comfort

AneticAid



QA3™
Patient Stretcher



The complete solution for patient transport

AneticAid

QA3™
Emergency Stretcher



Safe, robust and versatile patient stretcher – for the modern day emergency department

AneticAid

AT4™
Tourniquet Equipment



Ideal for single or bilateral orthopaedic surgery and pain management

AneticAid

Operating Theatre Accessories



Innovative solutions for you and your patients

AneticAid

For more information on any of our products or service contracts, call:
sales@aneticaid.com **+44 (0)1943 878647** aneticaid.com



SOCIAL VALUE



MADE IN THE UK



CARBON NEUTRAL

Innovative **Medical** Technology - **Practically Applied**



Anetic Aid Ltd, Sapper Jordan Rossi Park, Otley Road, Baildon BD17 7AX, UK